Municipalidad de Santa María La Paz
Email: munisantamaria@yahoo.com

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Novel coronavirus(SARS-CoV-2) IgG/IgM Antibody Combined Test Kit (Colloidal Gold Immunochromatographic Assay)

Please read this manual carefully before operation to ensure proper use

TEST KIT DESCRIPTION

The SARS-CoV-2 Rapid Test qualitatively detects IgG and IgM antibodies to SARS-CoV-2 in human serum, plasma or whole blood. The test applies colloidal gold method and is a tool to assist in the diagnosis of SARS-CoV-2 infection. For medical professional use only.

SUMMARY

In early January 2020, a novel coronavirus (SARS-CoV-2) was identified as the infectious agent causing an outbreak of viral pneumonia in Wuhan, China, where the first cases had their symptom onset in December 2019.¹ Coronaviruses are enveloped RNA viruses that are distributed broadly among humans, other mammals, and birds and that cause respiratory, enteric, hepatic, and neurologic diseases.² Six coronavirus species are known to cause human disease.³ Four viruses—229E, OC43, NL63, and HKU1 — are prevalent and typically cause common cold symptoms in immuno-competent individuals.³ The two other strains—severe acute respiratory syndrome coronavirus (SARS-COV) and Middle East respiratory syndrome coronavirus (MERS-COV)—are zoonotic in origin and have been linked to sometimes fatal illness.⁴

Common signs of infection include respiratory symptoms, fever, cough, shortness of breath and breathing difficulties. In more severe cases, infection can cause pneumonia, severe acute respiratory syndrome, kidney failure and even death.⁵

Standard recommendations to prevent infection spread include regular hand washing, covering mouth and nose when coughing and sneezing, thoroughly cooking meat and eggs. Avoid close contact with anyone showing symptoms of respiratory illness such as coughing and sneezing.

PRINCIPLE

Novel coronavirus(SARS-CoV-2) IgG/IgM Antibody Combined Test Kit is a colloidal gold-based method for detecting IgG and IgM antibodies against SARS-CoV-2 in whole blood, serum or plasma samples. new coronavirus recombinant antigen and chicken IgY were used as chromogenic markers on the binding pad of this test strip, and a mouse anti-human IgM monoclonal antibody was coated on the detection line (T) on the nitrocellulose membrane. And mouse anti-human IgG monoclonal antibody, the goat's anti-chicken IgY polyclonal antibody was coated at the quality control line (C). During the detection, when the sample to be tested contains IgM or IgG, it combines with the colloidal gold-labeled new coronavirus recombinant antigen to form an immune complex, which is captured and enriched by the reagent fixed on the membrane at the detection line (T) to form M Line or G line. The colloidal gold-labeled antibody diffused to the QC region and was captured by the secondary antibody to form a purple-red band in the QC region.

KIT CONTENTS

1. Reagent card: coated with new colloidal gold-labeled recombinant coronavirus recombinant antigen and chicken IgY glass fiber membrane, coated with mouse anti-human IgM monoclonal antibody and mouse anti-human IgG monoclonal antibody (solid phase) and sheep anti-chicken IgY nitrocellulose film, water absorption pad, PVC baseboard.

2. Sample dilution: 3mL / piece, containing phosphate, preservatives etc.
3. Detect components not required: timer
4. The components in different lot number are not interchangeable.

STORAGE AND STABILITY

The kit is stored in a dry and dark place at 4-30 ° C, and the validity period is 18 months (production date, expiration date or expiration date: see the label); the test reagent card is effective within 60 minutes after being exposed to a temperature of 25 ° C and a humidity of 40 to 65%.

PRECAUTIONS

Follow these instructions for the best results:

1. This kit is for in vitro diagnostic use only and cannot be reused.
2. Check the tightness of the aluminum foil bag before use. If there is a problem with the seal, do not use it.
3. Do not use expired reagents, do not mix reagents of different batches.
4. The test reagent card should be used immediately after opening, be careful not to get wet. Test reagent cards stored refrigerated in a refrigerator, and left to room temperature before opening for use.
5. Since whole blood samples are easily affected by factors such as hemolysis or blood stickiness, blood clots, it is recommended that serum / plasma samples be used.
6. Treat all samples and reaction waste as sources of infection.
7. When testing, please pay attention to protection. Wastes such as used test reagent cards are treated as biomedical waste.
8. Sample testing should be performed in a laboratory with certain conditions. All samples and materials in the testing process should be handled in accordance with the operating practices of infectious disease laboratories.

SAMPLE COLLECTION AND PREPARATION


- 1) This test card is suitable for testing whole blood, serum and plasma samples.
- 2) Whole blood and plasma samples can use EDTA, dipotassium, lithium heparin anticoagulant
- 3) Serum or plasma should be separated and tested as soon as possible after collection of clinical blood samples, and samples should not be left at room temperature for more than 4 hours.
- 4) Serum and plasma samples can be stored for 3 days at 2-8 ° C and 30 days below -20 ° C. Do not freeze and thaw repeatedly.
- 5) Whole blood samples can be stored for 3 days at 2 ~ 8 ° C, and should not be stored frozen. Hemolyzed samples should be discarded.
- 6) Before testing, the samples must be returned to room temperature and mixed thoroughly before use.

TEST PROCEDURE

* Note: Please read the instruction manual of this reagent carefully before all operations.


1. Restore the kit and sample to temperature (15 ~ 30 ° C).
2. Preparation: Open the packaging box, remove the test reagent card from the sealed aluminum foil bag, and lay it flat on a horizontal table.

- Aspirate 10 μ L of the sample (serum, plasma or whole blood sample) from the sample tube with a sample gun or a dropper.
- Drop the sample vertically into the sample hole on the test card. Immediately drop 2 drops (about 70 μ L) of the sample dilution liquid into the sample hole, and ensure that no air bubbles are generated during the operation. (Hint: the first 1-2 drops of sample dilution should be discarded, and then add the sample vertically to avoid air bubbles in the sample well)
- Timing, the result will be available within 15 minutes.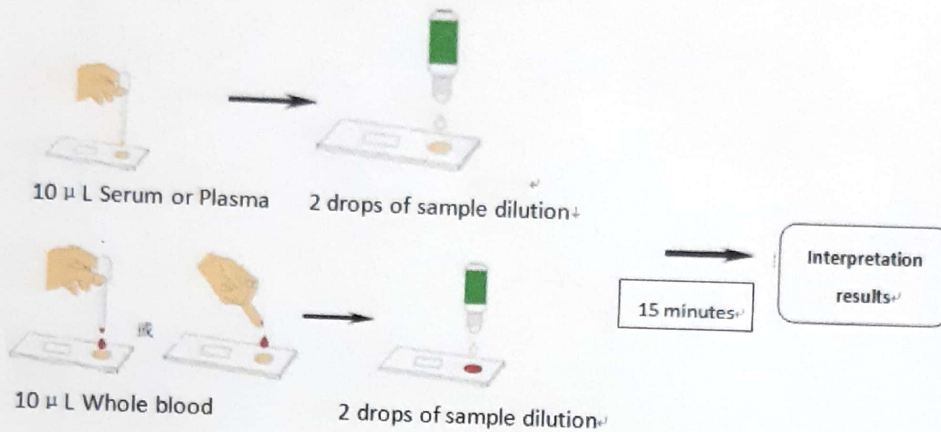


Interpretation of test results


- The test card only has a purplish red line at position C of the quality control line, indicating that no new IgM antibodies and IgG antibodies to the new coronavirus were detected, and the test result was negative.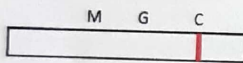


- A magenta line appears on the test card at the G position and the C control line, indicating that a new coronavirus IgG antibody was detected, and the test result was positive for IgG antibody.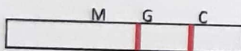


- A magenta line appears on the test card at the M position and the C control line, indicating that a new coronavirus IgM antibody was detected and the test result was positive for IgM antibody.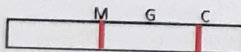

- A magenta line appears on the test card at the M position / G position and the C control position, indicating that the new coronavirus IgM antibody and IgG antibody were detected, and the test results were positive for IgM and IgG antibodies.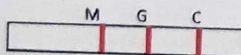

- The test card does not appear at the position C of the quality control line, no matter whether the test line appears at the position M or G, it is invalid, and it should be tested again.

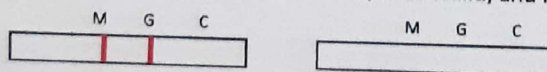LIMITATIONS

1. This kit only provides qualitative results.
2. The test should be completed within 15 minutes after the sample is added, and the measurement result beyond 15 minutes is invalid.
3. The positive test result of this reagent does not mean that there is no infection with pathogens other than the new coronavirus. This kit is only used for initial screening, and the test results cannot be used as a basis for diagnosis. In order to ensure the accuracy of the test results, other clinical examination methods can be used for further diagnosis.

PERFORMANCE CHARACTERISTICS

SENSITIVITY AND SPECIFICITY

1. Negative reference product compliance rate: The company's negative reference product is used for testing. The negative reference product compliance rate should be 20/20.
2. Positive reference product compliance rate: The company's positive reference product is used as the sample for testing. The IgG and IgM positive reference product compliance rate should be 10/10.
3. Sensitivity: Use the company's sensitivity reference as the sample for detection. The L1 sensitivity reference should be IgG antibody positive and IgM antibody negative. The L2 sensitivity reference should be positive for IgG and IgM antibodies.
4. Precision: 10 samples of test strips are randomly selected from the same batch of kits, and the company's precision reference is used as a sample for parallel testing. The test results should be consistent, all positive, and the color rendering uniform.
5. Lot difference: 10 samples of test strips were randomly selected from three batches of kits, and the company's precision reference product was used as a sample for parallel testing. The test results should be consistent, all positive, and uniform color.

CROSS-REACTIVITY

The Biopanda COVID-19 Rapid Test has been tested for anti-influenza A virus, anti-influenza B virus, anti-RSV, anti-Adenovirus, HBsAg, anti-Syphilis, anti-H. Pylori, anti-HIV and anti-HCV positive specimens. The results showed no cross-reactivity.

INTERFERING SUBSTANCES

The following compounds have been tested using the Biopanda COVID-19 Rapid Test and no interference was observed:

- Triglyceride: 50mg/dL
- Ascorbic Acid: 20mg/dL
- Hemoglobin: 1000mg/dL
- Bilirubin: 60mg/dL
- Total cholesterol: 6mmol/L

Label Introduce for user

Abbreviation	explanation	Abbreviation	explanation
	In vitro diagnostic medical device		Batch code
	Number of tests		Date of manufacture
	Manufacture		Use-by date
	EU Representative		Temperature: 4~30°C
	CE Label		Upright
	Caution		Keep dry
	Consult instructions for use		Fragile: handle with care
			Keep away from sunlight

BIBLIOGRAPHY

1. World Health Organization (WHO). WHO Statement Regarding Cluster of Pneumonia Cases in Wuhan, China. Beijing: WHO; 9 Jan 2020. [Accessed 26 Jan 2020]. <https://www.who.int/china/news/detail/09-01-2020-who-statement-regarding-cluster-of-pneumonia-cases-in-wuhan-china>
2. Weiss S R, Leibowitz J L. Coronavirus pathogenesis. *Adv Virus Res* 2011; 81:85-164. PMID: 22094080 DOI:10.1016/B978-0-12-385885-6.00009-2
3. Su S, Wong G, Shi W, et al. Epidemiology, genetic recombination, and pathogenesis of Coronaviruses. *Trends Microbiol* 2016; 24:490-502. PMID:27012512 DOI:10.1016/j.tim.2016.03.003
4. Cui J, Li F, Shi ZL. Origin and evolution of pathogenic coronaviruses. *Nat Rev Microbiol* 2019;17:181-192.PMID:30531947 DOI:10.1038/s41579-018-0118-9
5. World Health Organization (WHO). Coronavirus. <https://www.who.int/health-topics/coronavirus>
6. <https://www.biorxiv.org/content/10.1101/2020.02.07.937862v1.full.pdf>

Label for aluminum foil bag

1 Test/Kit

**Novel Coronavirus(SARS-CoV-2)IgG/IgM
Antibody Combined Test Kit**

	20200202	
	2021.08.10	

Yuno Diagnostics Co., Ltd.

Label for Reagent Kit



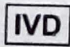
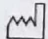
Novel
Coronavirus(SARS-CoV-2)
IgG/IgM Antibody
Combined Test Kit
(Colloidal Gold
Immunochromatographic
Assay)

	20200201
	2020.02.09
	2021.08.08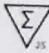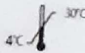


Label for carton

**Novel Coronavirus(SARS-CoV-2) IgG/IgM Antibody Combined
Test Kit(Colloidal Gold Immunochromatographic Assay)**

				20200201
				2020.02.09
				2021.08.08

Yuno Diagnostics Co., Ltd.
1,501, Building 3, Hongxin Industrial Park
.Guanguang Road1303., Guanlan Longhua
District, 518110, Shenzhen, Guangdong, P.R.
China

Wellkang Ltd
16 Castle Street, Dover, CT16
1PW, England, UK

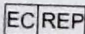 16 Castle Street, Dover, CT16
1PW, England, UK

SAN PEDRO SULA
 Col. Santa Ana, 11 y 12 Avenida
 12 Calle, Bloque No. 117, N.O
 Tel: (504) 2552-0511/ 2550-3850
 Email: ffloresprodylab@gmail.com

TEGUCIGALPA
 Bldr. Suyapa, frente a Hospital Escuela
 Edificio Plaza San José, local No. 1
 Tel: (504) 2235-5636/ 2231-1268
 Email: yriveraprodylab@gmail.com

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